

Taking measurements for arm stockings

Do not measure the patient until the best possible decongestion has been achieved. The results of treatment must be stabilised with appropriate bandaging until the arm stocking is supplied. In addition to your experience with measuring for flat knit garments, please also consider your client's degree of activity and circumstances.



When measuring circumferences, wrap the tape measure loosely and do **not** pull.

Arm circumferences
c - directly over the wrist



c¹ - at least 5 cm above **c**



d - middle of lower arm or thickest point of lower arm



e - over the elbow and the crook of the arm with the arm slightly bent



f - middle of the upper arm



g - 3 cm below the axilla



Mark all **length** measurements on the stretched inner arm and enter them on the measurement sheet.



g¹ - 3 cm above **g**









g¹-h - length for the shoulder-cap. Measure from "**g¹**" to the start of the bra-attachment or belt.







h-k-h - Measure the length for the shoulder-belt around and under the opposite axilla or between the breasts and right around the trunk. Start the measurement from the endpoint of the shoulder-cap.



medi armsleeves ccl. 1 and 2

Styles	 armsleeve	 combined	 armsleeve long	 combined long	 handpiece	 glove/mittens
Class 1	Item number Article Number					
	710 350500	730 350501	711 350510	731 350511	720 350600	
Class 2	712 360500	732 360501	713 360510	733 360511	722 360600	723 720800

Styles	 armsleeve with shoulder cap and belt	 armsleeve long with shoulder cap and belt	 armsleeve with hand + shoulder cap + belt	 armsleeve with hand long + shoulder cap + belt
Class 1	Item number Article Number			
	738 359500	739 359510	740 359501	741 359511
Class 2	742 369500	743 369510	744 369501	745 369511



standard sizes

Sizes	I	II	III	IV	V	VI	VII
ca	below 15	16 - 19	18 - 21	20 - 23	22 - 25	24 - 27	25 over
cc	below 13	13 - 15	15 - 17	17 - 19	19 - 21	21 - 23	23 over
ce	below 18	20 - 26	23 - 29	26 - 32	29 - 35	32 - 38	35 over
cg	below 21	22 - 29	25 - 32	29 - 36	33 - 40	37 - 44	41 over
	length C - G		standard < 44 cm		long > 44 cm		

