


mediven® thrombexin® 18

The standard anti-embolism stocking

Knee length (A-D)					
Size				Colour coding	
	Ankle (cB)	Calf (cC)	Thigh (cG)	Heel	Thigh
XS	17 – 19	26 – 32	–	orange	–
S	20 – 22	30 – 38	–	green	–
M	23 – 25	34 – 44	–	violet	–
L	26 – 28	38 – 48	–	blue	–
LX	26 – 28	44 – 55	–	grey	–
XL	29 – 31	44 – 55	–	red	–
Thigh length (A-G)					
XS	17 – 19	26 – 36	40 – 50	orange	orange
S	20 – 22	30 – 42	48 – 60	green	green
SX	20 – 22	30 – 42	56 – 70	green	grey
M	23 – 25	34 – 46	56 – 70	violet	violet
MX	23 – 25	34 – 46	64 – 80	violet	grey
L	26 – 28	38 – 51	64 – 80	blue	blue
LX	26 – 28	38 – 51	72 – 90	blue	grey
XL	29 – 31	42 – 55	72 – 90	red	red
XLX	29 – 31	42 – 55	80 – 100	red	grey
Thigh with waist attachment					
S	20 – 22	30 – 42	48 – 60	green	green
M	23 – 25	34 – 46	56 – 70	violet	violet
L	26 – 28	38 – 51	64 – 80	blue	blue
LX	26 – 28	38 – 51	72 – 90	blue	grey




Where to measure

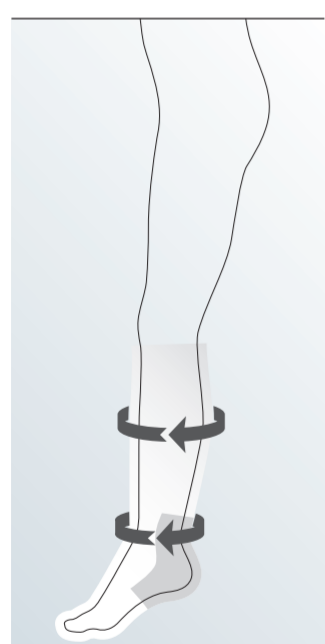

Below knee

The stocking size (centimetres) is determined by the ankle measurement (circumference at b). Calf circumference measurement (c) confirms the stocking size.

Calf circumference at widest point. Ankle circumference at narrowest point.

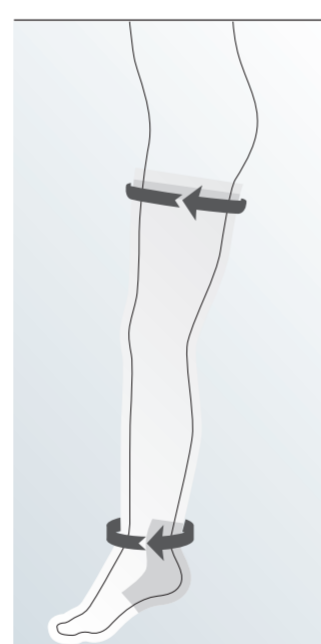

Thigh length

The stocking size (centimetres) is determined by the ankle measurement (circumference at b). Thigh circumference measurement (g) confirms the stocking size.

Thigh circumference at intended position of top band. Ankle circumference at narrowest point.

Intelligent silicone top band on thigh length stockings.



correct



incorrect

medi hospital butler



medi hospital butler

remove medi hospital butler/valet
Pull the medi hospital valet off downwards and pull the stocking up to the groin.

medi GmbH & Co KG
Medicusstraße 1
95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-781
export@medi.de
www.medi.de



medi. I feel better.